HUMANTECHNIK

*lisa*  
soundmonitor

**DE**

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Seite 2

**EN**

**User manual**

Page 10

**FR**

**Mode d'emploi**

Page 18

**NL**

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Pagina 26

**IT**

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Pagina 34

**ES**

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Página 42

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

**Figure 1: View of top / side / front**

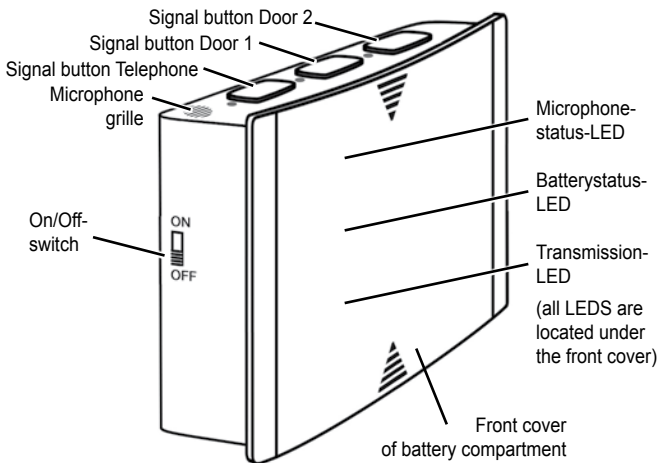**Figure 2: View of underside**

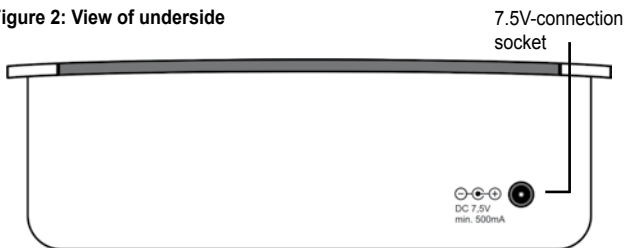Congratulations on purchasing your »*lisa soundmonitor*«.

You have chosen a highly modern and reliable system. Please read this manual thoroughly. It will tell you how to set up the system correctly and explain all the features this device has to offer.

### Package contents

Please check if the following parts are included in the package:

- »soundmonitor«
- 3 x NiMH rechargeable battery, type AA, 1,2V (already inserted in the compartment)
- 7,5 V power supply unit
- Wall mount
- User manual
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

### Safety instructions

- Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full.
- Store this user manual in a safe place so that it is also accessible for other users at any time.
- When using power tools always follow the instructions of the manufacturer and wear suitable protective equipment (e.g. protective goggles).

- Kindly check before starting to drill if there are any hidden electric cables or water pipes in the walls. In case of doubt we recommend using a cable detector.

### Function principle

The »soundmonitor« has been designed to learn sounds and to recognise them if they occur again. Up to three different sounds can be memorized.

This makes it possible to bind special acoustic signals into the »*lisa RF*« alerting system. The »soundmonitor« scans the environmental noise around it for those sounds it has been taught to react to.

If a memorised sound has been recognised, the »soundmonitor« will send out a radio signal to all »*lisa*«-receivers within range.

The receivers indicate the radio signal with light-, vibration- or an acoustic alert (this depends upon the type of receiver).

Please note that the sound recognition only works with exactly repeatable sounds.

## Inserting / replacing the rechargeable batteries (included and already inserted)

Remove the battery compartment lid: Push the front plastic cover upwards. A certain resistance, caused by the latching, has to be overcome. (Figure 3)

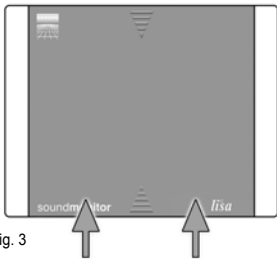
Fig. 3

The plastic cover can now be taken off, the battery compartment has become accessible. Observe the correct polarity when inserting / replacing the batteries (Figure 4).

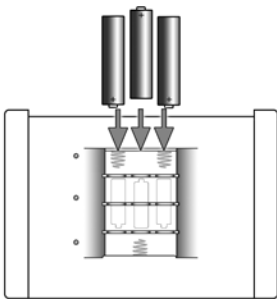
Fig. 4

Attach the plastic cover: Lay the cover onto the housing with approx 1 cm distance to the lower edge (Figure 5).

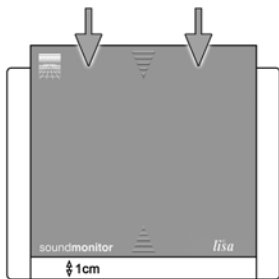
Fig. 5

Push the cover downwards until it clicks back into place. The front cover is now attached again.

## Indicators and controls

3 indicator LEDs are located under the dark transparent front cover:

### Microphone status indicator

This LED lights up whenever a random sound triggers the activation of the microphone.

### Battery status indicator

A red battery status LED indicates that the rechargeable batteries are almost depleted and must be recharged. During recharging, this LED will show an orange light. A permanent green light shows that the batteries are charged and the »soundmonitor« is active.

## Transmission indicator

This LED indicates any outgoing transmission with a red light.

## Signal buttons

Function and transmission range can be tested with the three signal buttons (»Telephone«, »Door 1« and »Door 2«). The signal buttons also serve to start the sound-teaching procedure.

## Setup

### 1. Choosing the best position

The »soundmonitor« scans and analyses the noises in its vicinity for the patterns of the sounds it has been taught.

In order to assist this function, the »soundmonitor« should be placed as close as possible to the sound source it is supposed to identify (for example placed on a wall underneath the door bell).

In order to ensure the largest possible transmission range, please note the following recommendations:

- The transmitter should not be placed close to interference-emitting devices such as TV sets, computer screens and mobile phones.
- The transmitter should not be placed on metal objects or surfaces due to the radio-shielding effect of the metal.

### 2. Teaching the desired sounds

Switch the »soundmonitor« on by shifting the On/Off switch to the »ON« position.

To teach a new sound, press and hold one of the signal buttons, located on the upper face of the device, for 2 to 3 seconds until the adjacent LED of the button starts to blink.

Trigger the desired sound on the sound source. The sound should have a length of one to two seconds.

During the recording, the microphone status indicator must be lit constantly!

The LED of the signal button will light up and go out after the sound has been recorded.

The new sound has been taught. The other two signal buttons can also be programmed as described above.

Please note that the identified sounds will be displayed on the receivers according to the symbol of the signal buttons (»Telephone«, »Door 1« and »Door 2«).

#### Important notice:

When teaching a sound, please avoid any other environmental noise.

The soundmonitor should remain in the position in which the according sound was recorded, in order to assist a secure recognition of the sound when it occurs again.

#### Please note:

This device can only recognise repeatable and unaltering electronic sounds. A mechanical sound source or voice cannot be identified.

### **3. Power supply**

Connect the power supply with the according socket on the underside of the »soundmonitor«.

Plug the power supply unit into a regular 230V mains electricity outlet.

### **Blackout-Backup**

Please note that this device has been designed to be primarily powered over the power supply unit.

The rechargeable batteries serve only as an emergency backup during power cuts.

The standby provided by the batteries covers approx. 16 hours.

The batteries are charged when the »soundmonitor« is connected to the mains power supply and the ON/OFF switch is in »ON« position.

The batteries have a charging time of approx. 12 hours.

### **4. Setting the correct radio channel**

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

The »soundmonitor« is set to channel 0 by default.

It is only necessary to set units to other channels if you have several different users operating »lisa RF« systems in the same building.

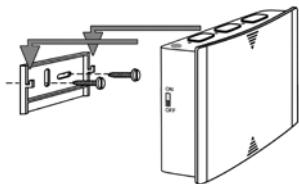
Set the required channel by using a small screwdriver to turn the channel selector switch, which is located on the underside of the device, to the required channel number.

All transmitters and receivers must be set to the same channel.

## Mounting the »soundmonitor«

The »soundmonitor« can be mounted using the supplied wall mount. To use it, fix the wall mount onto a wall (or rafter, cupboard etc.) When mounting onto a brick- or stone wall, you may need a power drill and stone bit to do so. Use the supplied wall plugs in this case.

Once fixed, hang the »soundmonitor« onto the wall mount bracket.

## Maintenance and care

The »soundmonitor« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Technical data

|                              |   |
|------------------------------|---|
| Power supply:                | 7,5 V-power supply unit on 110 - 230 V, 50-60 Hz, power backup by 3 rechargeable batteries 1,2V |
| Rech. battery standby:       | approx. 16 hours  |
| Frequency:                   | 868,35 MHz  |
| Temperature range:           | 0° C – 40° C  |
| Microphone status indicator: | 1 red LED   |
| Battery status indicator:    | 1 red/green LED   |
| Transmission indicator:      | 1 red LED   |
| Recording indicators:        | 3 red LEDs  |
| Height:                      | 95 mm   |
| Width:                       | 112 mm  |
| Depth:                       | 38 mm   |
| Weight:                      | 320 g (including batteries)   |

This device complies with the requirements of the following EC directives:

- 2011 / 65 / EC RoHS-directive
- 2002 / 96 / EC WEEE-directive
- 2014 / 53 / EC RED-directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humantechnik.com/service](http://www.humantechnik.com/service).

Technical specifications subject to change without prior notice.

## Warranty

The »soundmonitor« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

## Rechargeable batteries


The included rechargeable batteries can be recycled. Please discard them through  the battery container or a specialist shop. Please discard only discharged batteries to assist environmental protection.


**Disposal** of used electrical and electronic units (applicable in the countries of the  European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product.

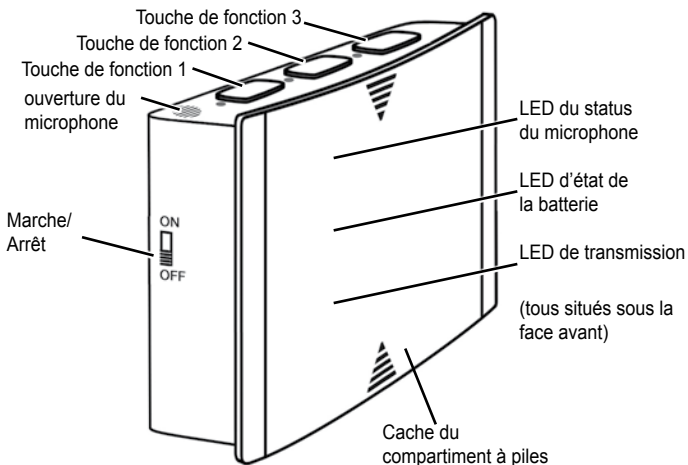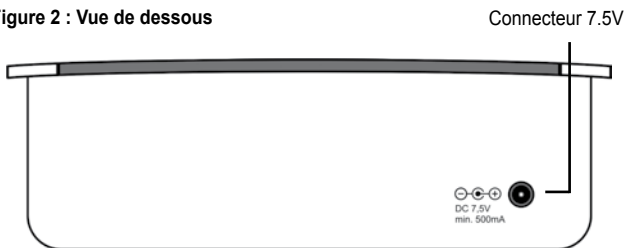Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

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**Sarabec Ltd**