AMPLICALL 10

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
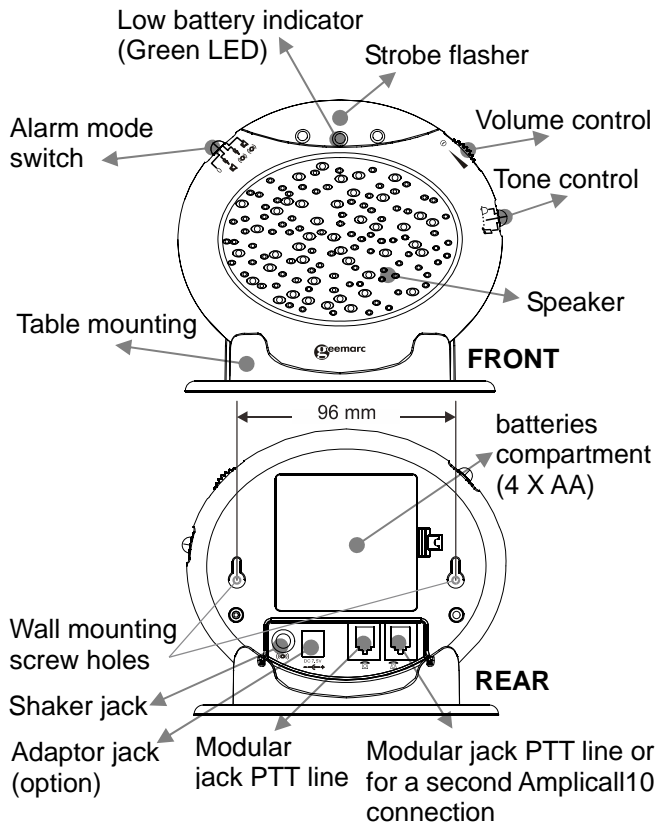
INSTALLATION

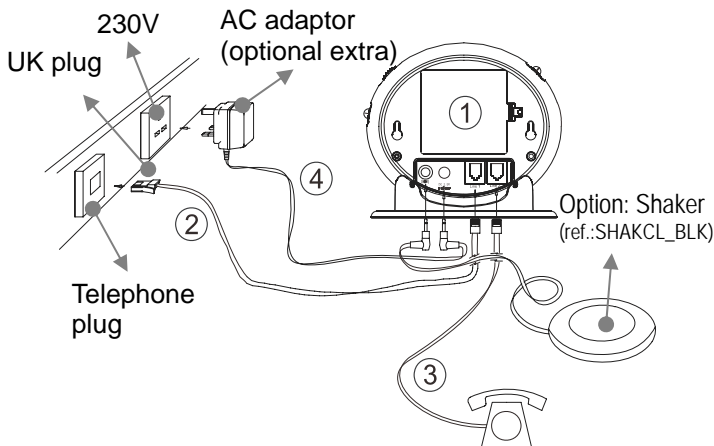Installation (See drawing)

- 1) Insert 4 AA batteries into the battery compartment (optional extra)
- 2) Connect the telephone line cord with the BT connection and RJ11 connection from the PTT socket on the wall to the jack found at the rear of the Amplicall 10
- 3) Connect the RJ11 to RJ11 cable to the line jack found at the rear of the Amplicall 10 and to your telephone.
- 4) Plug the power cable into the power adapter jack found at the rear of the Amplicall 10. Plug the adaptor into the mains supply at the wall (optional extra)

INSTALLATION

Electrical connection : The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as «hazardous voltage» according to EN60950 standard). The apparatus does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the apparatus, ensure that the mains power socket is readily accessible.

Telephone connection : Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

INSTALLATION

Alarm Operation

1 - SETTING THE ALARM MODE

Choose which alarm mode you want by moving the alarm mode switch to one of the following positions:

0 : no alarm

☒ both strobe flasher and sound alarm activated

⚡ : both strobe flasher and shaker (if connected)
activated

☒ : both sound alarm and shaker (if connected)
activated

2 - VOLUME CONTROL SLIDE SWITCH

Set the volume of the alarm sound or power on/off.

3 - TONE CONTROL SWITCH (3 POSITIONS)

Set the tone of the alarm sound.

INSTALLATION

Wall Mounting

1. Drill two holes horizontally, 96mm apart, and leave the screws protruding from the wall by 5mm.
2. Hang the base on to the two screws and pull the base down to lock it into place.

Table Mounting

If you want to mount the unit on a table, simply place it in the base cradle and place the base cradle on the table.

Change the Battery

The unit requires 4 x AA batteries

1. Open the battery compartment door.
2. Insert 4 x AA batteries into the battery compartment.
3. Snap the battery compartment door back into place.

Note: when the green LED indicator flashes, it is time to replace the batteries.

Whilst replacing the batteries, disconnect all telephone line cords from the modular wall jacks.

Second Amplicall 10

Another Amplicall 10 can be added to the system (call 01707 387602)

INSTALLATION

Trouble shooting

Symptom	Solution
No incoming call yet the Amplicall10 rings automatically or rings continuously; the shaker vibrates automatically or continuously; the strobe LED flashes automatically.	<ol style="list-style-type: none">1) Check the batteries are inserted correctly.2) Check the power adaptor is connected correctly.3) Turn off the power switch for 10 minutes and then turn it on.
The red low battery LED flashes automatically.	<ol style="list-style-type: none">1) Replace the batteries.2) Turn off the power switch for 10 minutes and then turn it on.
When there is an incoming call but no ringing can be heard.	<ol style="list-style-type: none">1) Check it is connected to the power adaptor correctly;2) Check the batteries are installed correctly.3) Check if the volume switch is switched on.4) Check it is connected to the line cord correctly.
When there is an incoming call but the shaker doesn't vibrate.	<ol style="list-style-type: none">1) Check it is connected to the shaker correctly.2) Check the shaker socket to see if it has been damaged.3) Check the alarm mode selection switch position is correct.

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our helpline or visit our website at **www.geemarc.com**. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM

Please note : The guarantee applies to the United Kingdom only.

Declaration : Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE and in particular article 3 section 1a, 1b and section 2.

*The declaration of conformity may be consulted at **www.geemarc.com***

GUARANTEE

Recycling directives : The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products so at the end of their useful life are recycled in the best way. When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it. Thus if you respect these instructions you ensure human health and environmental protection.

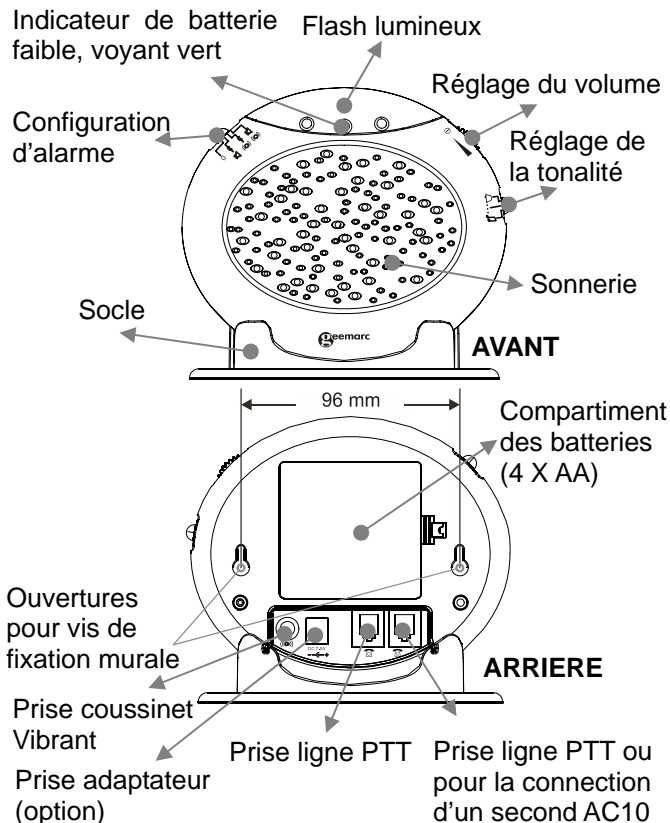
For product support and help visit our website

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